



WHITEPAPER

MOBILITY Lab
BY APDM

Mobility Lab provides sensitive, valid and reliable outcome measures.

With hundreds of universities and hospitals using this system worldwide, Mobility Lab is the most trusted wearable gait and balance system on the market.

Using APDM's advanced wearable sensors (Opals), Mobility Lab makes it easy to collect, analyze, and store outcome measures. Attach sensors to your subject, and instruct them to perform a standardized test. A report is then automatically generated to compare against normative values. This process takes less than five minutes.

What Can You Measure?

GAIT

Lower Limb

- Cadence
- Foot Clearance
- Gait Cycle Duration
- Gait Speed
- Double Limb Support
- Lateral Step Variability
- Lateral Swing Max
- Pitch at Initial Contact
- Pitch at Toe Off
- Stance
- Step Duration
- Stride Length
- Swing
- Toe Out Angle

Upper Limb

- Maximum Velocity
- Range of Motion

BALANCE

Trunk

- Coronal Range of Motion
- Saggital Range of Motion
- Transverse Range of Motion

Lumbar

- Coronal Range of Motion
- Saggital Range of Motion
- Transverse Range of Motion

Head

- Coronal Range of Motion
- Saggital Range of Motion
- Transverse Range of Motion

Postural Sway

- 95% Ellipse Sway Area
- RMS Sway
- Coronal RMS Sway
- Saggital RMS Sway

Turning

- Angle
- Duration
- Maximum Velocity

Sit to Stand

- Lean Angle
- Duration

Stand to Sit

- Lean Angle
- Duration

DEFINITIONS

Lower Limb

Cadence	The number of steps per minute, counting steps made by both feet
Gait Cycle Duration	The duration of the gait cycle (one left plus right step duration)
Gait Speed	The forward distance (2 step lengths) travelled during the gait cycle divided by the gait cycle duration
Foot Clearance	The height of the sensor on the foot, relative to its start position during standing and measured at mid-swing
Double Support	The percentage of the gait cycle in which both feet are on the ground
Lateral Step Variability	In three consecutive steps, the perpendicular deviation of the middle foot placement from the line connecting the first and the third step
Lateral Swing Max	The amount that the foot travels perpendicular to forward movement while swinging forward during an individual stride
Pitch at Initial Contact	The dorsiflexion of the foot at initial contact (typically heel strike)
Pitch at Toe Off	The plantar flexion of the foot just as it leaves the floor at push off
Stance	The percentage of the gait cycle in which the foot is on the ground
Step Duration	The duration of a step
Stride Length	The forward distance travelled by a foot during a gait cycle
Swing	The percentage of the gait cycle in which the foot is not on the ground
Toe Out Angle	The lateral angle of the foot during the stance phase, relative to the forward motion of the gait cycle

Upper Limb

Maximum Velocity	The maximum rotational velocity of the arm swing
Range of Motion	The angular range of the arm swing

Sternum Range of Motion

Coronal	The angular range of the thoracic spine in the coronal plane (roll)
Sagittal	The angular range of the thoracic spine in the sagittal plane (pitch)
Transverse	The angular range of the thoracic spine in the transverse plane (yaw)

Lumbar Range of Motion

Coronal	The angular range of the lumbar spine in the coronal plane (roll)
Sagittal	The angular range of the lumbar spine in the sagittal plane (pitch)
Transverse	The angular range of the lumbar spine in the transverse plane (yaw)

Sit To Stand

Duration	The duration of the sit to stand transition
Lean Angle	The angular range of motion of the trunk during the sit to stand transition

Stand To Sit


Duration	The duration of the stand to sit transition
Lean Angle	The angular range of motion of the trunk during the stand to sit transition

Turning

Angle	The rotational angle of the turn
Duration	The duration of the turn
Velocity	The peak angular velocity of the turn

Postural Sway

95% Ellipse Sway Area	The area of an ellipse covering 95% of the sway angle in both the coronal and sagittal planes
RMS Sway	The extent of postural sway calculated as the root mean square (RMS) of the sway angle in both the coronal and sagittal planes
Coronal RMS Sway	The extent of postural sway calculated as the root mean square (RMS) of the sway angle in the coronal plane
Sagittal RMS Sway	The extent of postural sway calculated as the root mean square (RMS) of the sway angle in the sagittal plane

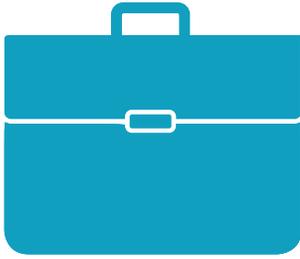


PORTABLE

Set up in any location with our lightweight, wireless system




RELIABLE

Numerous clinical studies have proven high test-retest reliability

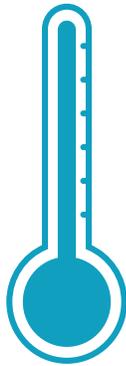

SENSITIVE

Accurately measure minimally detectable changes

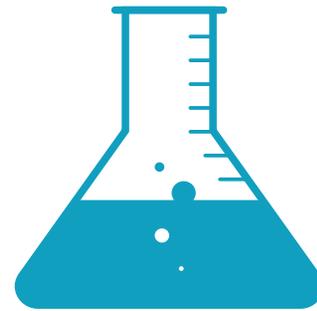

VALID

Algorithms validated against video motion capture and force plate systems


PUBLICATIONS

- Simoes. **"Feasibility of Wearable Sensors to Determine Gait Parameters."** University of South Florida. 2011
- Mancini, et al. **"Mobility Lab to Assess Balance and Gait with Synchronized Body-worn Sensors."** *Bioengineering & Biomedical Science*. Emerging Technology for Use in Rehabilitation Issue. 2012
- Spain, et al. **"Body-worn Motion Sensors Detect Balance and Gait Deficits in People with Multiple Sclerosis who have Normal Walking Speed."** *Gait & Posture*. 2012
- Beach. **"Effect of Compliant Flooring on Postural Stability in an Older Adult Population and in Individuals with Parkinson's Disease."** University of Dayton. 2013.
- Fling, et al. **"Asymmetric Pedunculopontine Network Connectivity in Parkinsonian Patients with Freezing of Gait."** *Brain - A Journal of Neurology*. 2013
- King, et al. **"Exploring Outcome Measures for Exercise Intervention in People with Parkinson's Disease."** *Parkinson's Disease*. 2013
- Martori, et al. **"Knee Angle Analysis Using a Wearable Motion Analysis System for Detection and Rehabilitation of Mild Traumatic Brain Injury."** University of South Florida. 2013
- Martori. **"A Wearable Motion Analysis System to Evaluate Gait Deviations."** University of South Florida. 2013
- Mirelman, et al. **"VTIME: A Treadmill Training Program Augmented by Virtual Reality to Decrease Fall Risk in Older Adults."** *BMC Neurology*. 2013
- Balasubramanian. **"Age Related Changes in Balance and Gait."** Arizona State University. 2014
- Baston, et al. **"Postural Strategies Assessed with Inertial Sensors in Healthy and Parkinsonian Subjects."** *Gait & Posture*. 2014
- Chaikereee, et al. **"Interaction of Age and Foam Types Used in Clinical Test for Sensory Interaction and Balance (CTSIB)."** *Gait & Posture*. 2014
- Dewey, et al. **"Automated Gait and Balance Parameters Diagnose and Correlate with Severity in Parkinson Disease."** *Journal of the Neurological Sciences*. 2014
- El-Gohary, et al. **"Continuous Monitoring of Turning in Patients with Movement Disability."** *Sensors*. 2014
- Fling, et al. **"Functional Reorganization of the Locomotor Network in Parkinson Patients with Freezing of Gait."** *PLOS One*. 2014
- Horak & Mancini. **"Objective Biomarkers of Balance and Gait for Parkinson's Disease Using Body-worn Sensors."** *Movement Disorders*. 2014
- Horak, et al. **"Role of Body-Worn Movement Monitor Technology for Balance and Gait Rehabilitation."** *Physical Therapy*. 2014
- Howell, et al. **"Dual Task Gait Balance Control Assessment with an Inertial Measurement Unit Following Concussion."** University of Oregon. 2014
- King, et al. **"Instrumenting the Balance Error Scoring System for Use with Patients Reporting Persistent Balance Problems After Mild Traumatic Brain Injury."** *Archives of Physical Medicine and Rehabilitation*. 2014
- Mancini, et al. **"Quantifying Freezing of Gait in Parkinson's Disease During the Instrumented Timed Up and Go Test."** *IEEE Eng Med Biol Soc*. 2014
- Pearson, et al. **"Turn Detection and Characterization with Inertial Sensors."** *Sensors*. 2014
- Peterson, et al. **"Dual-Task Interference and Brain Structural Connectivity in People with Parkinson's Disease who Freeze."** *Movement Disorders*. 2014
- Smith, et al. **"Consistency in Administration and Response for the Backward Push and Release Test: A Clinical Assessment of Postural Responses."** *Physiotherapy*. 2014
- Wang, et al. **"Inertial Measurements of Free-Living Activities: Assessing Mobility to Predict Falls."** *IEEE*. 2014
- Weiss, et al. **"Long-term Outcome of Deep Brain Stimulation in Fragile X-associated Tremor/Ataxia Syndrome."** *Parkinsonism and Related Disorders*. 2014
- Cohen, et al. **"Lighten Up: Specific Postural Instructions Affect Axial Rigidity and Step Initiation in Patients with Parkinson's Disease."** *NeuroRehabilitation and Neural Repair*. 2015
- Coulthard, et al. **"Evaluation of an Inertial Sensor System for Analysis of Timed-Up-and-Go Under Dual-Task Demands."** *Gait & Posture*. 2015
- Curtze, et al. **"Levodopa is a Double Edged Sword for Balance and Gait in People with Parkinson's Disease."** *Movement Disorders*. 2015
- Mancini, et al. **"Continuous Monitoring of Turning in Parkinson's Disease: Rehabilitation Potential."** *NeuroRehabilitation*. 2015
- McConnell & Silverman. **"Comparing Usability and Variance of Low and High Technology Approaches to Gait Analysis in Healthy Adults."** University of Nevada. 2015


CONTENTS

GAIT CYCLE ANALYSIS	1	POSTURAL TRANSITION MEASURES	8
BODY PLANES	2	• Turning Analysis	
SENSOR CONFIGURATIONS AND MEASURES	3	Turn Angle	
MOBILITY LAB TEST MEASURES	4	Turn Duration	
		Turn Velocity	
		• Sit To Stand Analysis	
		Lean Angle	
		Duration	
		• Turn To Sit Analysis	
		Turn duration	
		Turn Velocity	
		• Trunk Range Of Motion	
FULL BODY GAIT MEASURES	5	POSTURAL SWAY MEASURES	10
• Gait Cycle Analysis		• Postural Sway	
Stance		95% Ellipse Sway Area	
Swing		95% Ellipse Rotation	
Gait Cycle Duration		RMS Sway	
Gait Speed		Mean Velocity	
• Double Support			
• Spatial Analysis		MOBILITY LAB FOOTPLATE	11
Cadence			
Step Duration			
Stride Velocity			
Stride Length			
Toe Out Angle		TESTS	12
• Lateral Step Variability		• Walk Test	
• Circumduction		• Timed Up and Go Test	
Lateral Swing Max		• Postural Sway Test	
• Clearance		• mCTSIB Test	
• Heel Pitch		• BESS Test	
• Toe Pitch		• mBESS Test	
• Arm Range Of Motion		• 360° Turn Test	
Arm Swing Velocity		• 5x Sit To Stand	

GAIT CYCLE ANALYSIS

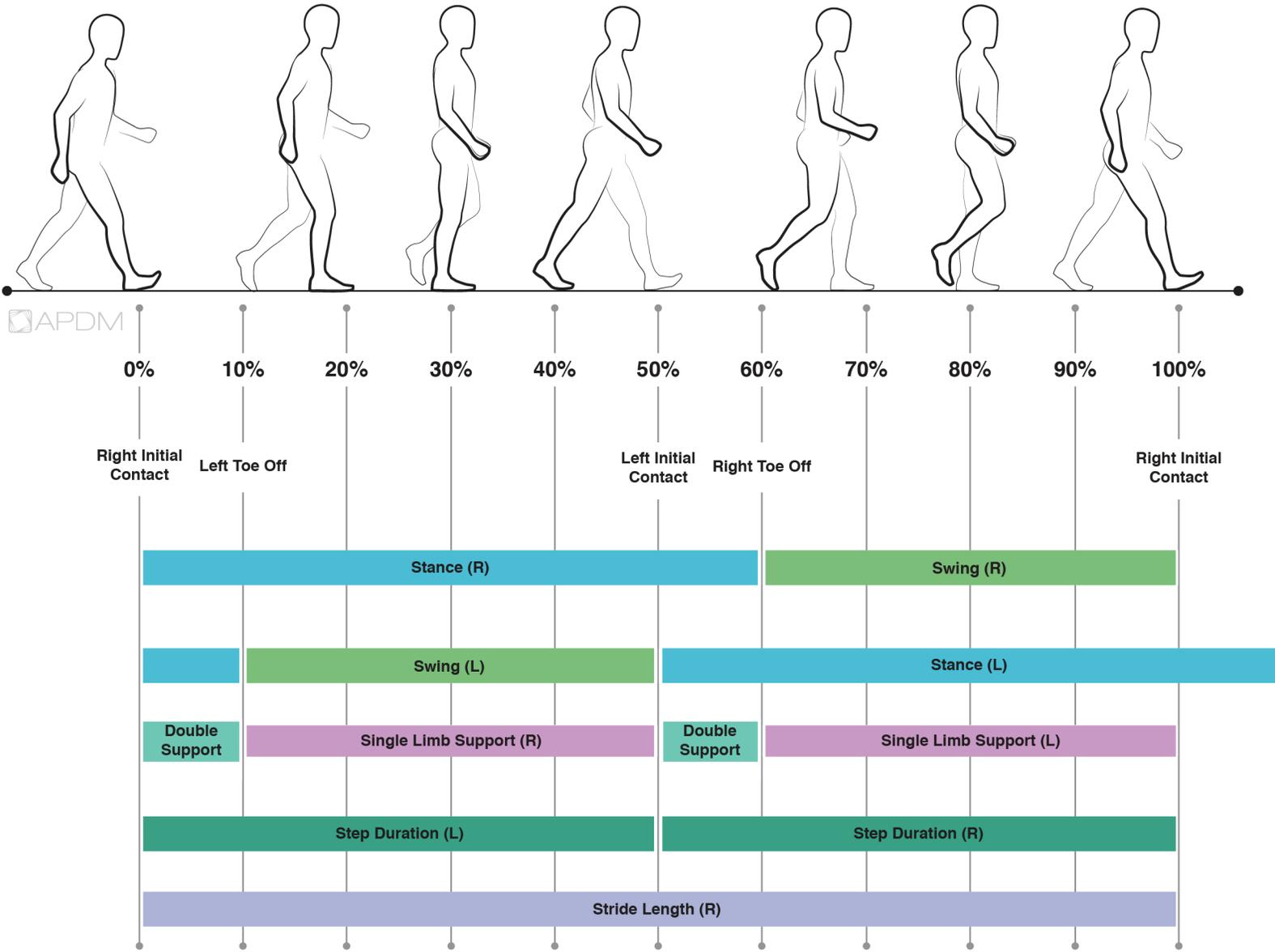


BODY PLANES

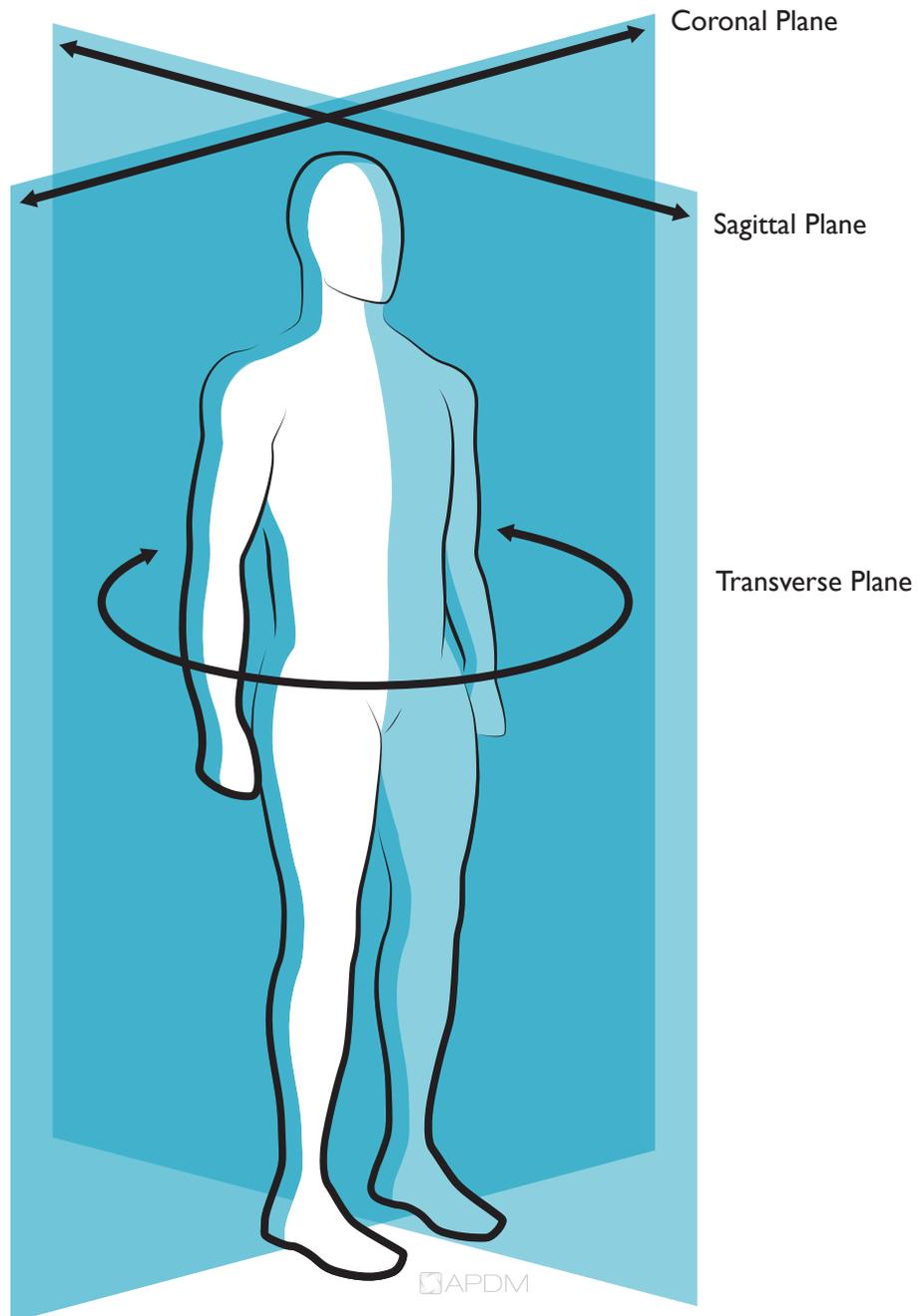


SENSOR CONFIGURATIONS AND MEASURES

BALANCE

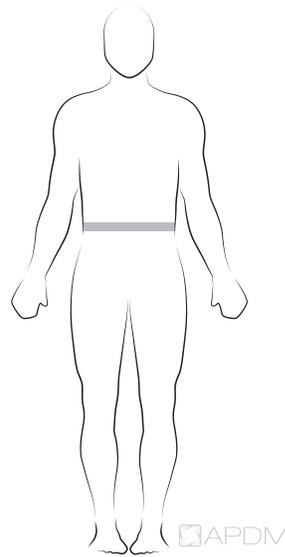


1 OPAL

BALANCE, LOWER LIMB GAIT, TURNING

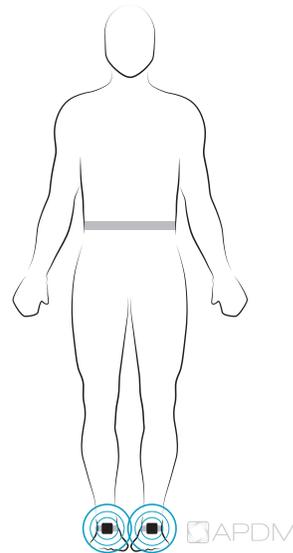

3 OPALS

BALANCE, LOWER LIMB GAIT, UPPER LIMB GAIT, TURNING, SIT TO STAND

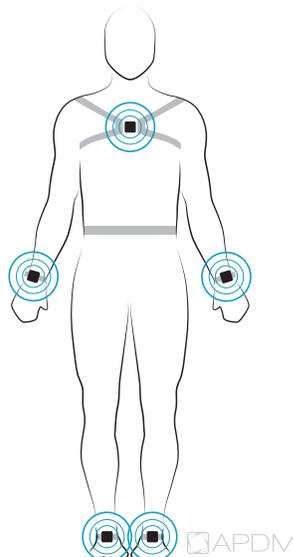

6 OPALS

MOBILITY LAB TEST MEASURES

TESTS


-  Walk
-  TUG
-  Sway
-  CTSIB
-  BESS
-  mBESS
-  360° Turn
-  Sit to Stand

	Tier 1 1 Opal sensor	Tier 2 3 Opal sensors	Tier 3 6 Opal sensors
Lower Limb			
Cadence		W	W
Gait Cycle Duration		W	W
Gait Speed		W	W
Foot Clearance		W	W
Double Support		W	W
Lateral Step Variability		W	W
Circumduction		W	W
Foot Strike Angle		W	W
Toe Off Angle		W	W
Stance		W	W
Step Duration		W	W
Stride Length		W	W
Swing		W	W
Toe Out Angle		W	W
Upper Limb			
Maximum Velocity		W	W
Range of Motion		W	W
Sternum Range of Motion			
Coronal			W
Sagittal			W
Transverse			W
Lumbar Range of Motion			
Coronal		W	W
Sagittal		W	W
Transverse		W	W
Sit To Stand			
Duration			sS
Lean Angle			sS
Stand To Sit			
Duration			sS
Lean Angle			sS
Turning			
Angle		W T	W T 360
Duration		W T	W T 360
Velocity		W T	W T 360
Steps in Turn		W T	W T 360
Postural Sway			
95% Ellipse Sway Area	S	S C B mB	S C B mB
RMS Sway	S	S C B mB	S C B mB
Coronal RMS Sway	S	S C B mB	S C B mB
Sagittal RMS Sway	S	S C B mB	S C B mB


FULL BODY GAIT MEASURES

Gait measures are detected, analyzed, and averaged over the extent of the walking duration of the subject. All measures are assessed for asymmetry and variability.


GAIT CYCLE ANALYSIS

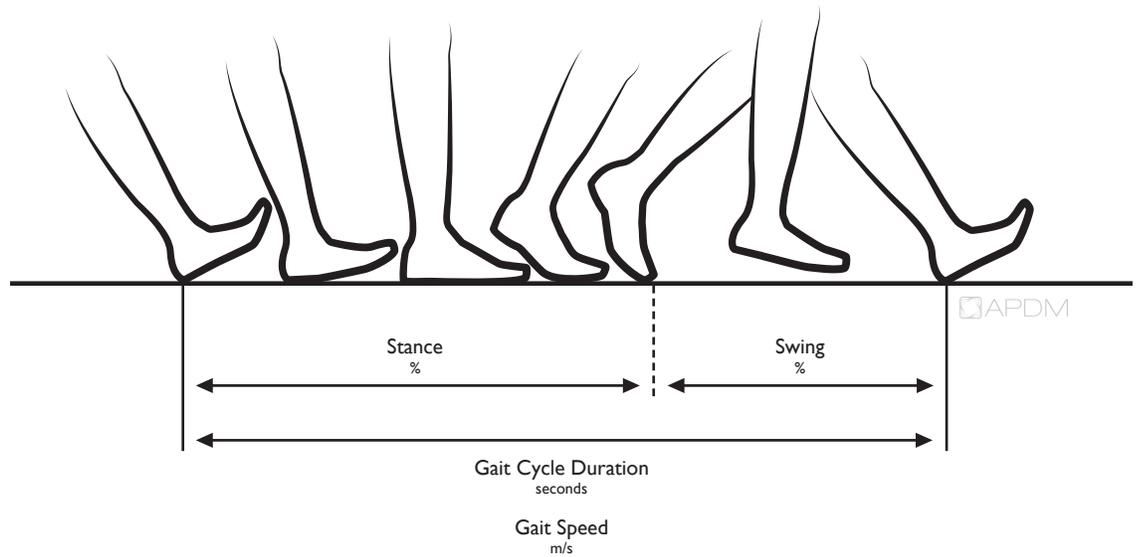


DOUBLE SUPPORT

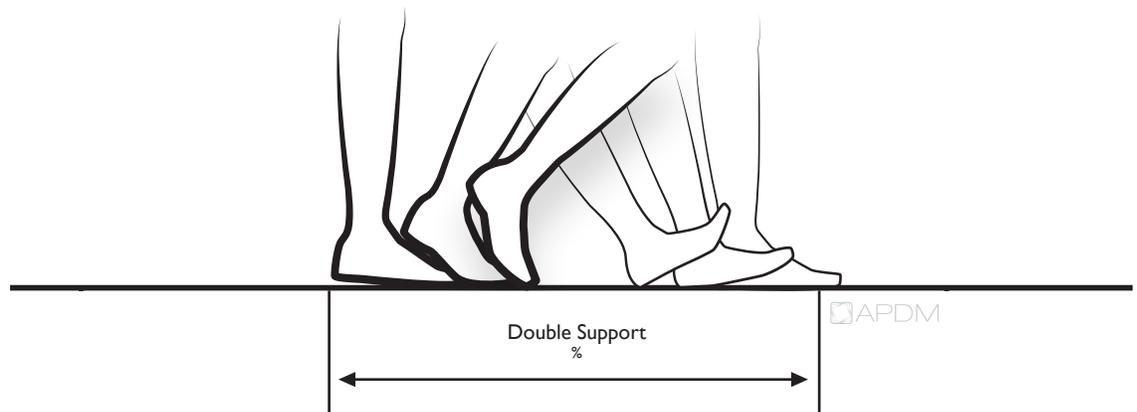


TERMINAL DOUBLE SUPPORT

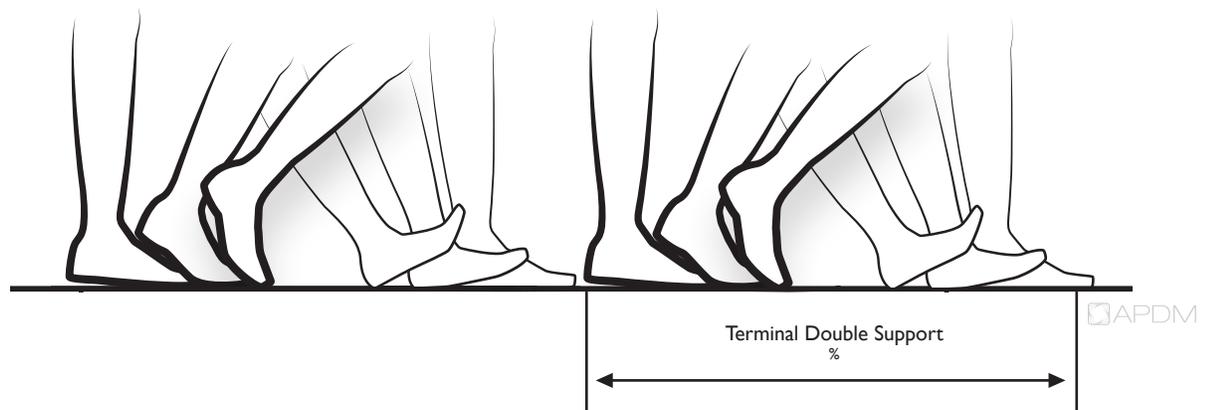


SPATIAL ANALYSIS

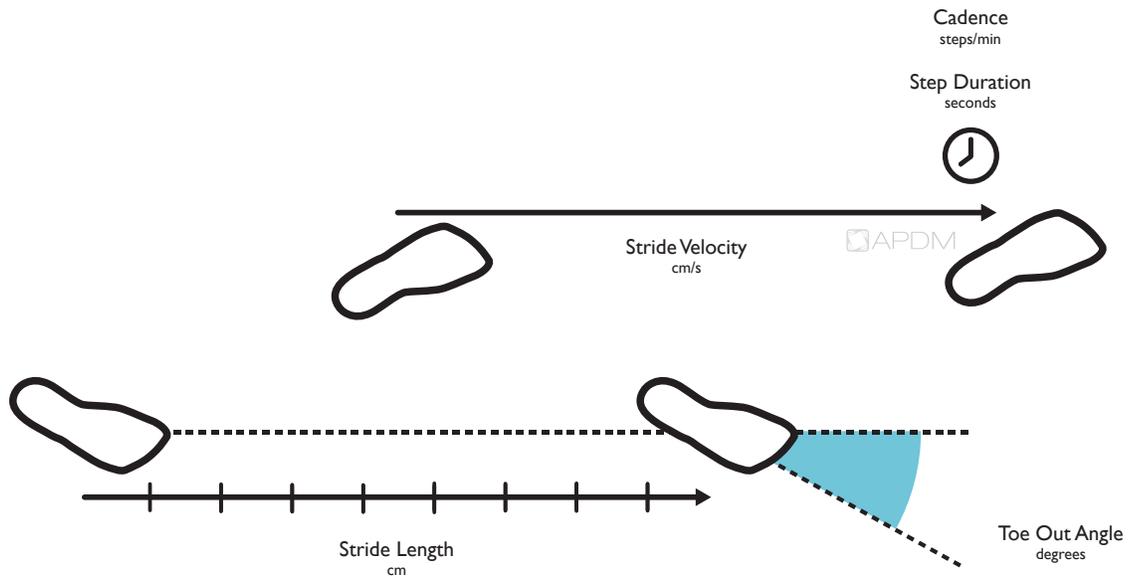


LATERAL STEP VARIABILITY

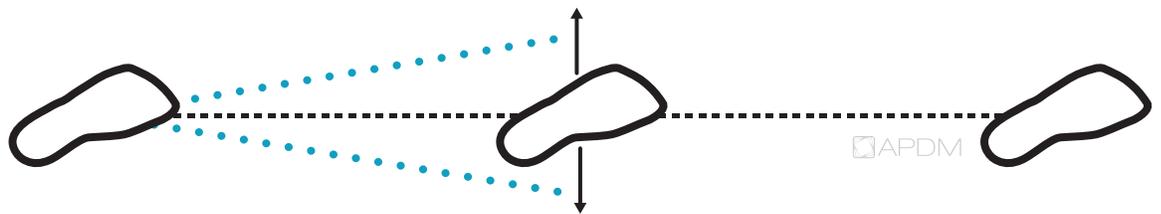


CIRCUMDUCTION

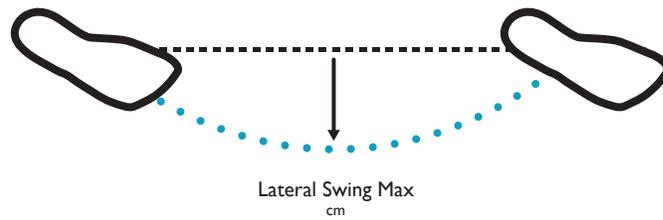


CLEARANCE

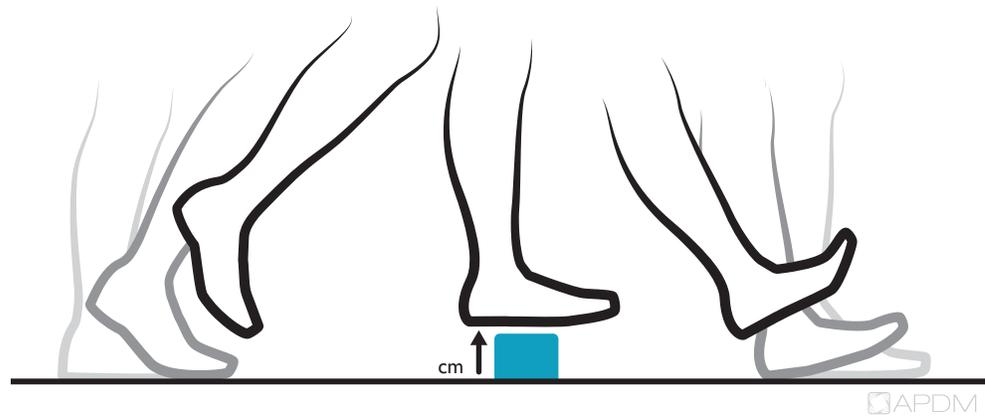

HEEL/TOE PITCH

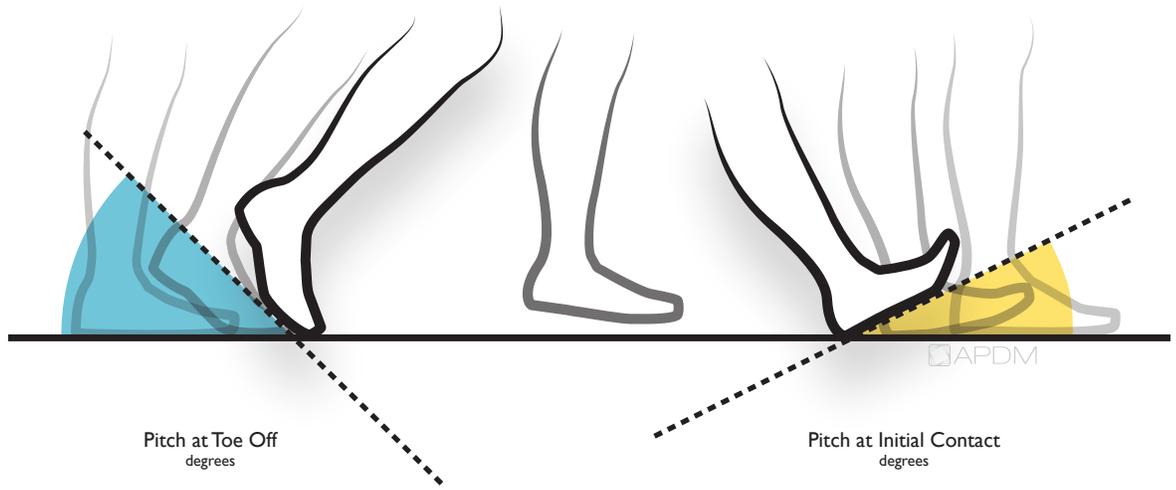


ARM RANGE OF MOTION

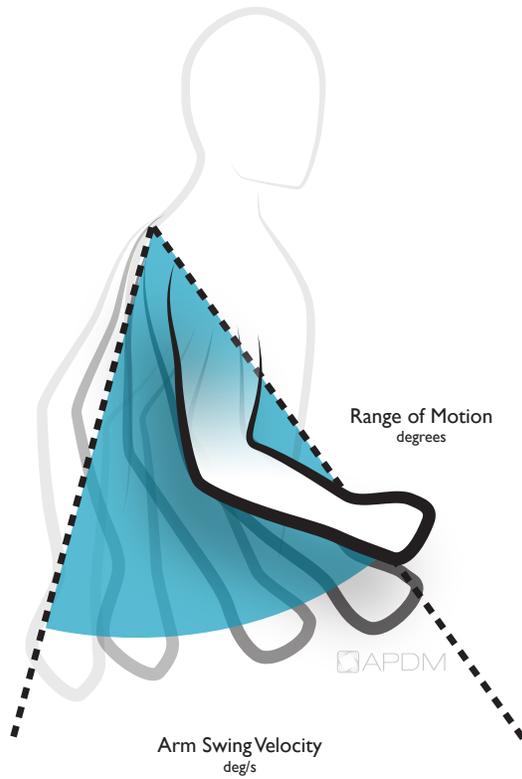


POSTURAL TRANSITION MEASURES

Postural transitions are detected, analyzed, and averaged over the extent of the walking duration of the subject.


TURNING ANALYSIS

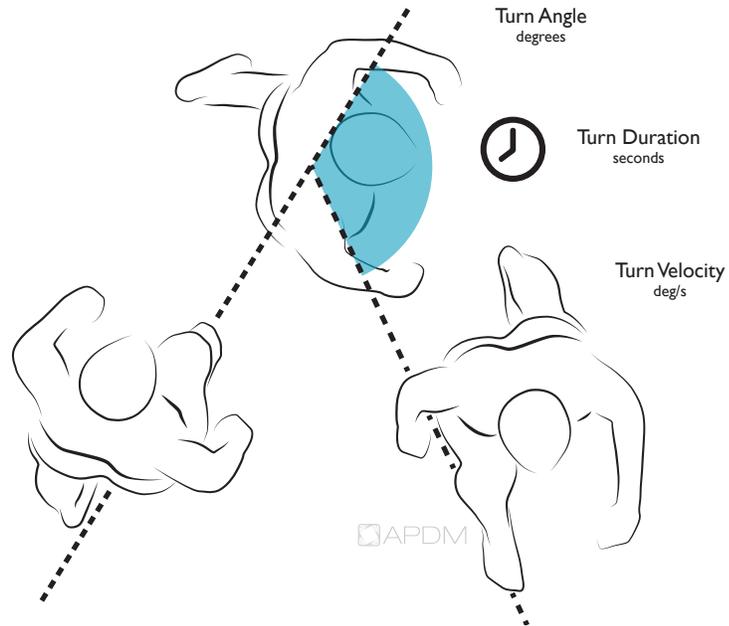

SIT TO STAND ANALYSIS

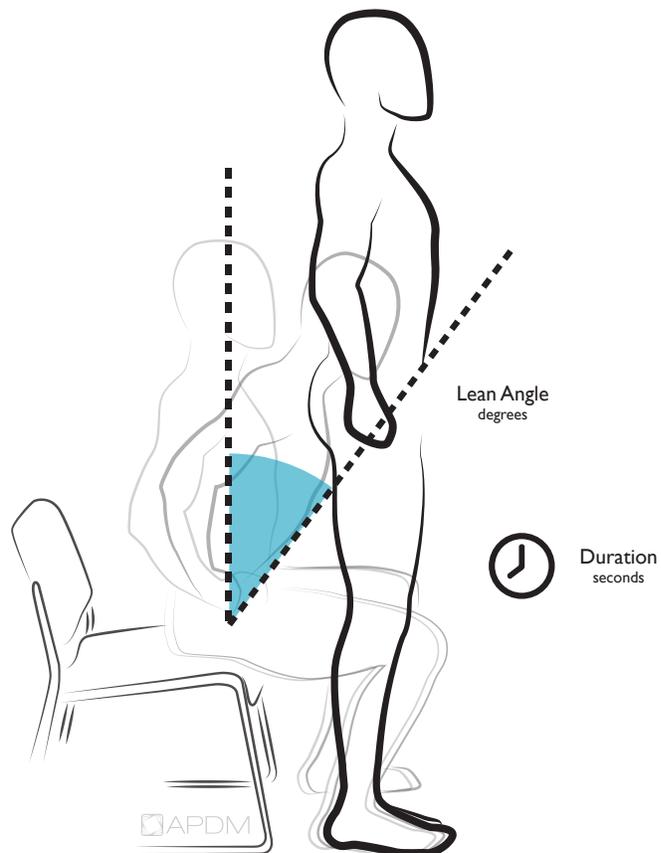

TURN TO SIT ANALYSIS

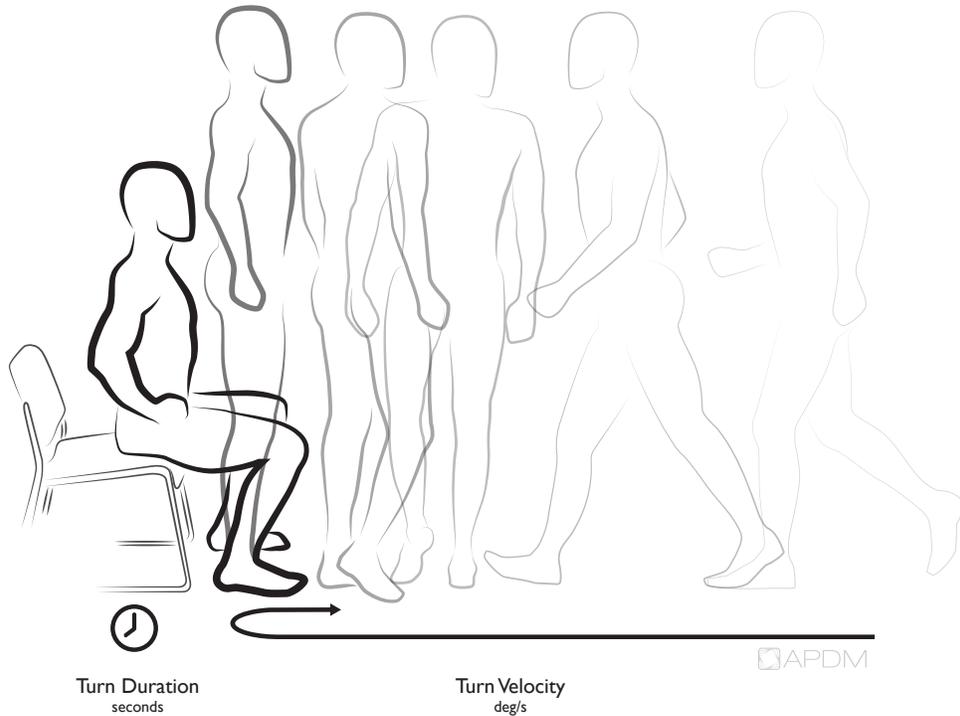


Sagittal Range of Motion
degrees

Transverse Range of Motion
degrees


Coronal Range of Motion
degrees

TRUNK RANGE OF MOTION

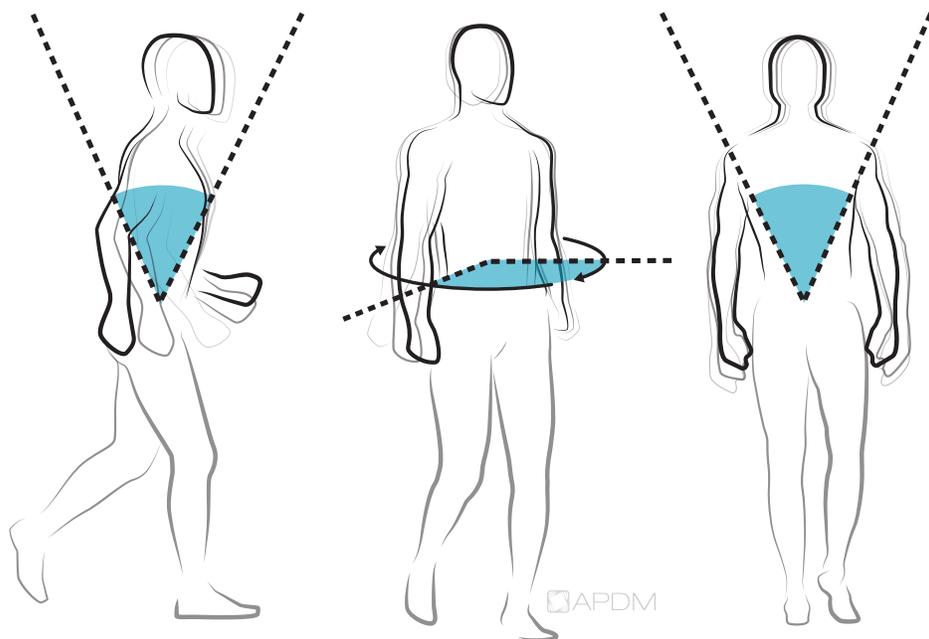

POSTURAL SWAY MEASURES


All postural sway measures are assessed using the Opal movement sensor placed on a subject's lumbar. All metrics are reported in Coronal, Sagittal and Transverse planes.

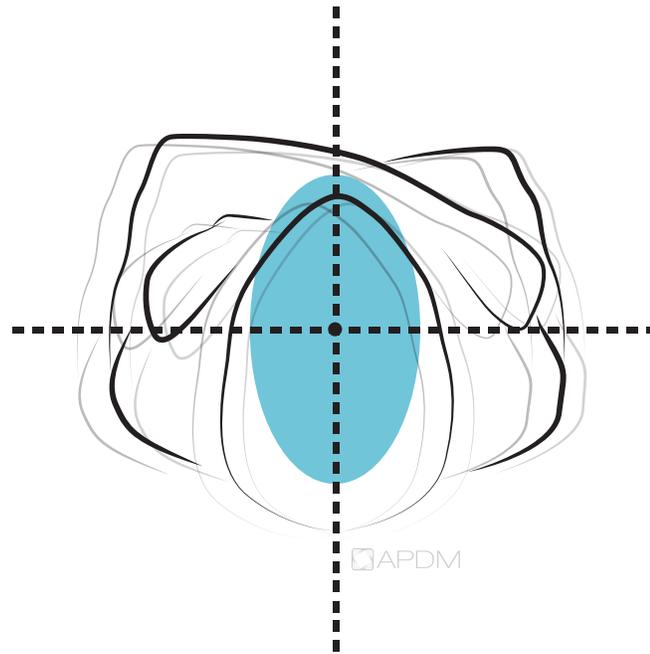

95% Ellipse Sway Area
degrees²

95% Ellipse Rotation
degrees

POSTURAL SWAY

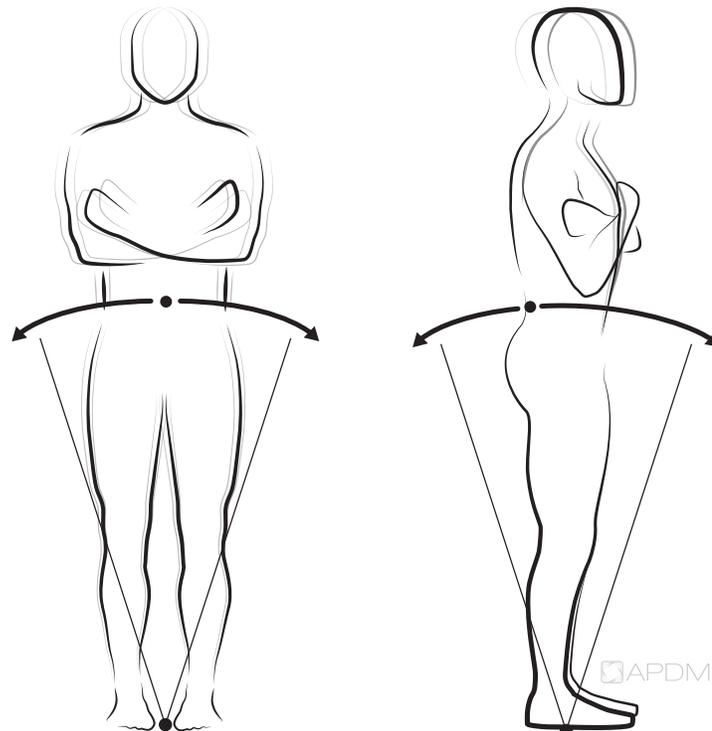


RMS Sway
m/s²

Mean Velocity
m/s³


FOOTPLATE

The Mobility Lab Footplate is designed to standardize stance width for each Mobility Lab test. All norms are derived from subjects using the standardized stance width measured by the Footplate. Standard instructions for some tests instruct the subject stand with their feet together to induce instability, but research has shown that using Mobility Lab with a wider stance is equally as sensitive and puts the subject at less of a risk of falling during the tests.

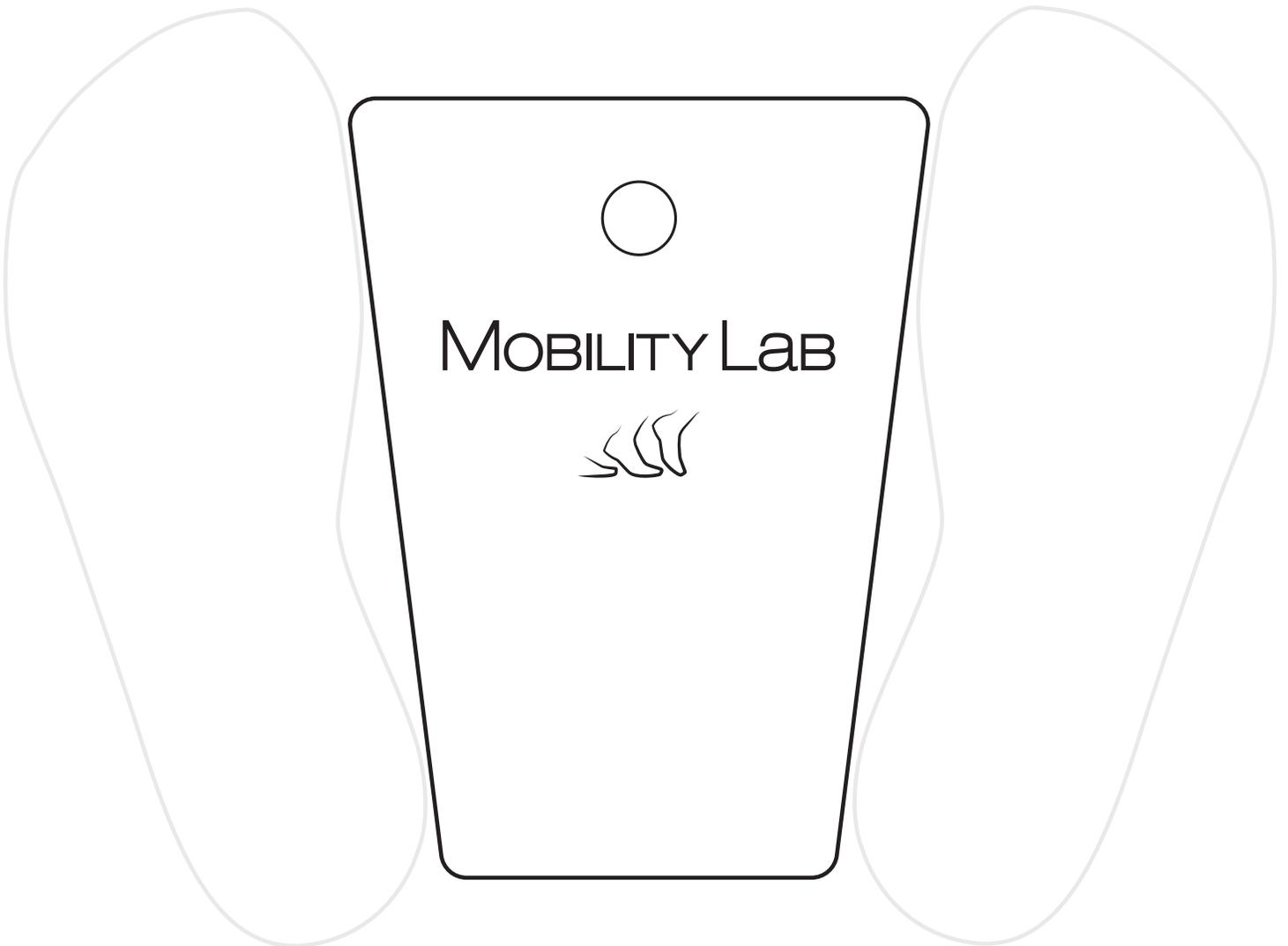

WALK TEST

The Walk test is the most comprehensive test to measure a subject's gait. We recommend that your subject walks for at least 2 minutes in order to collect a sufficient amount of gait cycles to accurately measure variability and asymmetry.

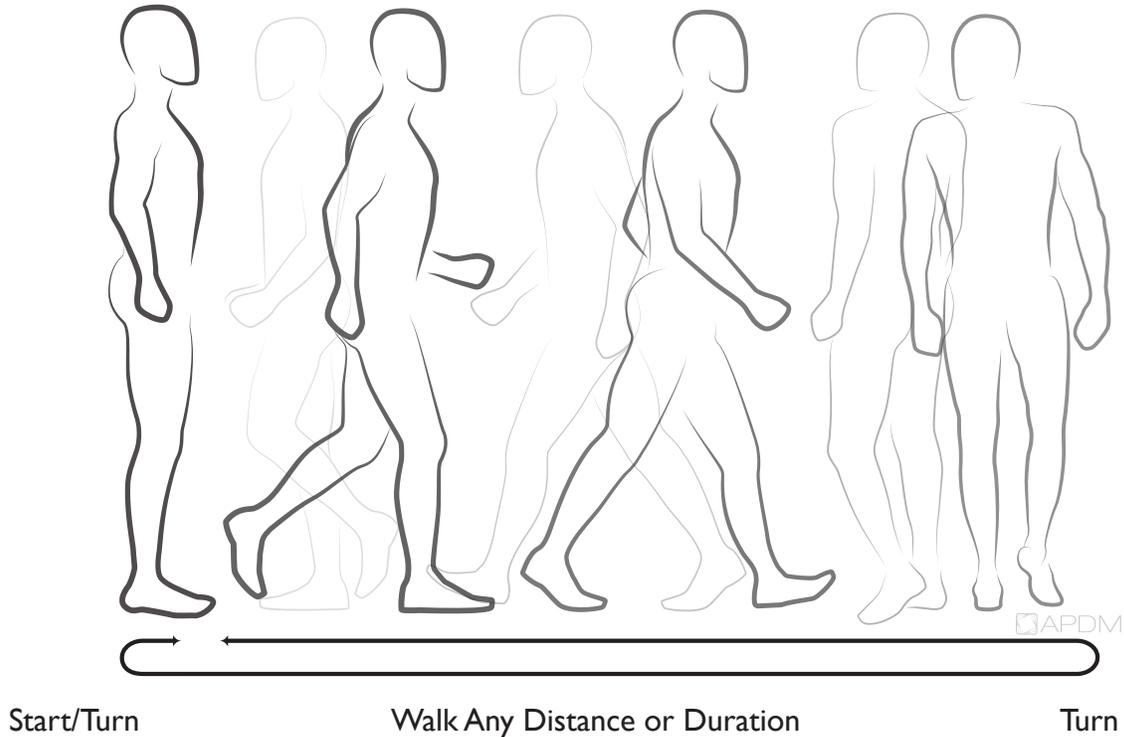

TEST MEASURES:

Full body gait (legs, arms, and trunk), asymmetry, variability and turning

NUMBER OF SENSORS: 3 or 6

SETUP:

1. Walking corridor must be at least 7 meters in length

PROTOCOL:


1. Select Walk and press start trial.
2. Subject should stand comfortably and wait for instruction to begin walking.
3. When the subject is ready, press record and instruct the subject to walk.
4. The subject can walk freely in a straight path and preform 180 degree turns when necessary.
5. Terminate the trial at any point.

NORMATIVE VALUES:

Normative values were collected using a 2 minute walk in a corridor 7 meters or longer with 180 degree turns at both ends.

TIMED UP AND GO TEST

Timed Up and Go (TUG) is a common test to assess a subject's mobility. APDM has made it more valuable by giving you the ability to precisely measure all of the components of mobility, rather than just duration.

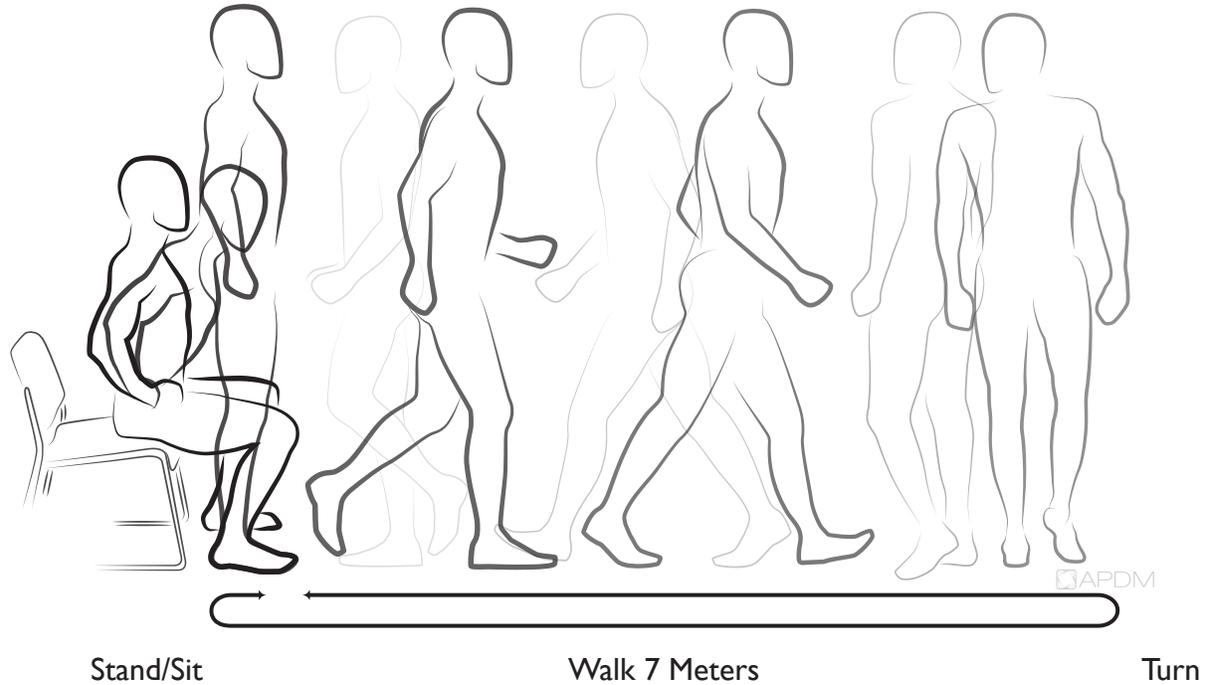

TEST MEASURES:

Postural transitions (sit, stand, and turning)

NUMBER OF SENSORS: 3 or 6

SETUP:

1. Measure 7 meters, placing tape at the two ends.
2. Place an armless chair at the start before the tape.

PROTOCOL:


1. Select TUG and press start test.
2. Subject should sit comfortably in the chair with their arms on their legs, and back against the seat.
3. When the subject is ready, press record and the test will begin to count down from 3 seconds.
4. The subject should rise from the chair without using their arms and begin walking.
If the subject is unable to rise from the chair with arms, reset the test and allow them to use their arms to stand.
5. After the subject walks passed the 7m end tape, they should turn 180 degrees and walk back.
6. Once they arrive at the chair they should turn 180 degrees, and sit down.
7. Terminate the trial when the subject rests their back against the back of the seat.

NORMATIVE VALUES:

Normative values were collected using the protocol listed above.

POSTURAL SWAY

The instrumented Postural Sway (Sway) test is a common test of quiet stance balance. It is a very simple test comprising of only one sensor on the lumbar. The test takes only 30 seconds to administer.

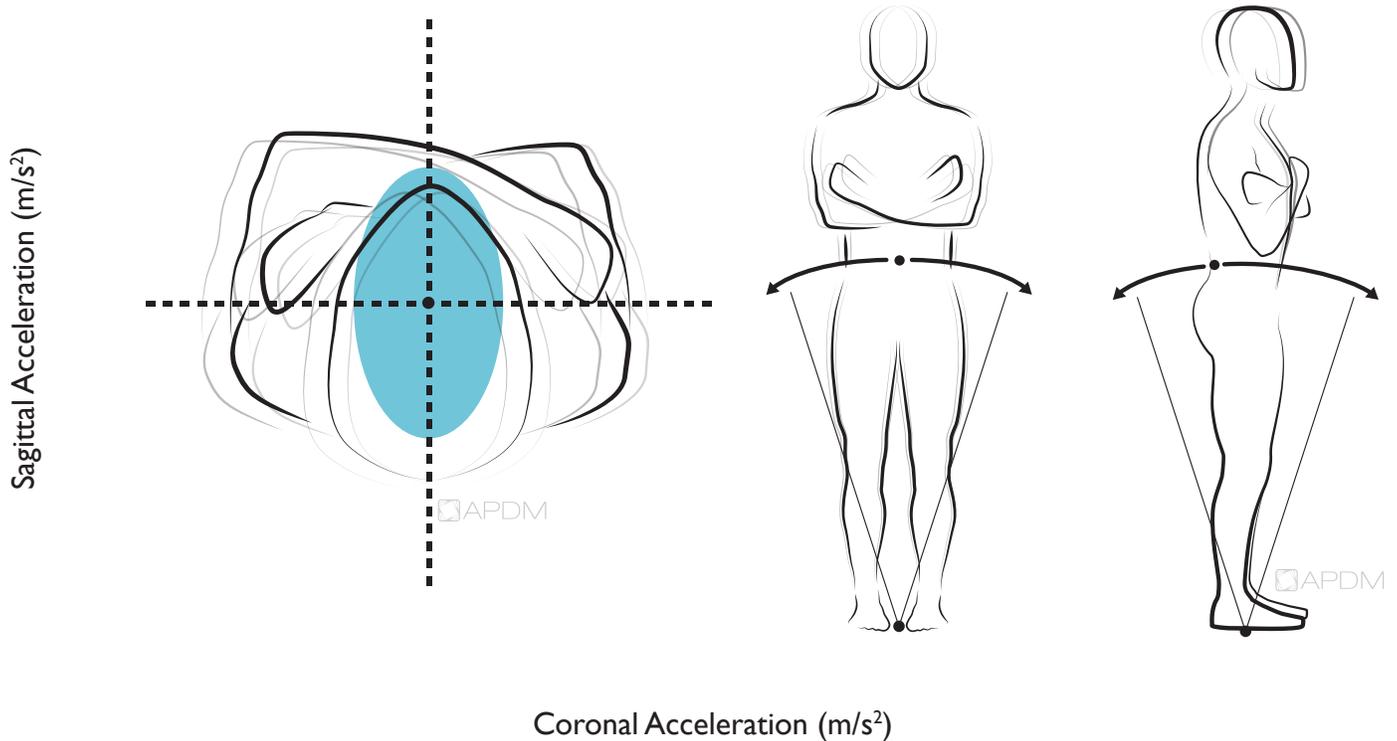

TEST MEASURES:

Postural sway

NUMBER OF SENSORS: 1 or 3 or 6

SETUP:

1. Have the subject fit their feet around the foot template provided with the Mobility Lab system (to normalize foot placement).

PROTOCOL:


1. Select Sway and press start test.
2. Subject should stand comfortably with their hands at their side or across their chest.
3. Press start and wait for the test to count down from 30 seconds.

NORMATIVE VALUES:

Normative values were collected with eyes open on a hard surface with arms crossed.

mCTSIB

The modified Clinical Test of Sensory Interaction and Balance (mCTSIB) is a composite test to assess a subject's balance under different test conditions.

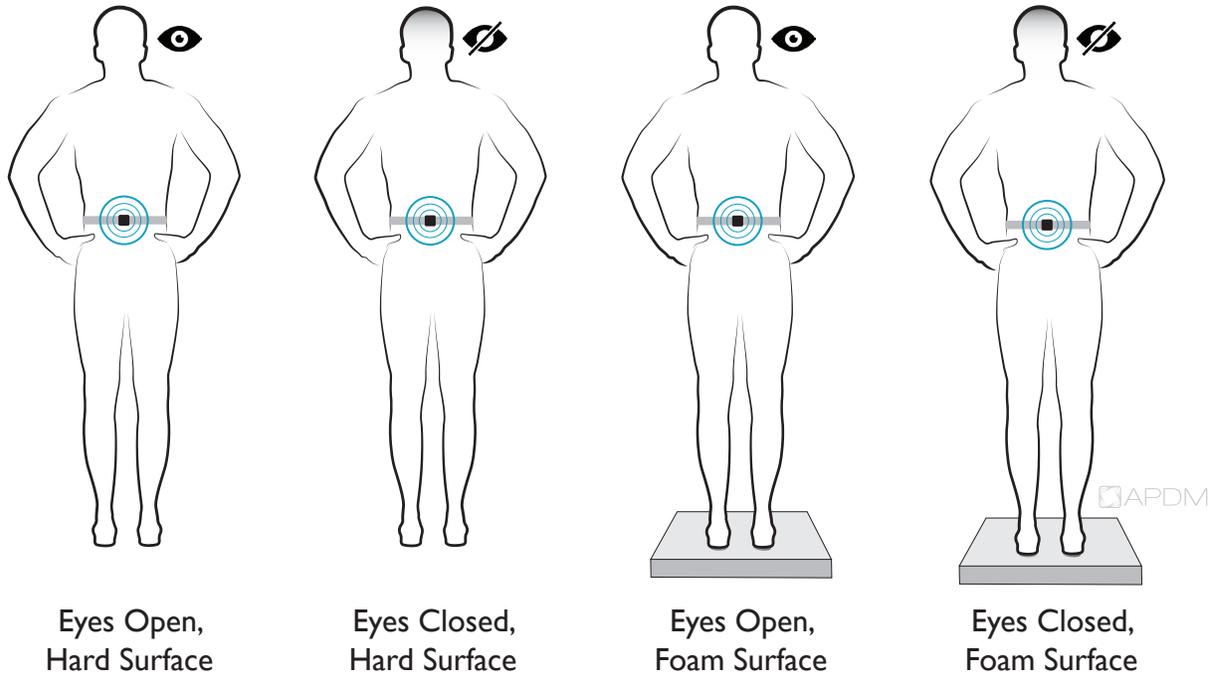

TEST MEASURES:

Postural sway, visual dependence, proprioceptive dependence, and vestibular loss

NUMBER OF SENSORS: 1 or 3 or 6

SETUP:

1. Have the subject fit their feet around the foot template provided with the Mobility Lab system (to normalize foot placement).

PROTOCOL:

1. Select CTSIB and press start test.
2. Subject should stand comfortably with their feet together and hands at their side.
3. Follow the conditions outlined in the test description.
4. Press start and wait for the test to count down from 30 seconds. Move on to the next test condition.

TEST CONDITIONS:


1. Eyes Open, Hard Surface
2. Eyes Closed, Hard Surface
3. Eyes Open, Foam Surface
4. Eye Closed, Foam Surface

NORMATIVE VALUES:

Normative values were collected following the protocol listed above.

BESS

The Balance Error Scoring System (BESS) test is a measure of assessing static postural stability. It is designed for the mild head injury population, and to assist in return to sports play decisions.

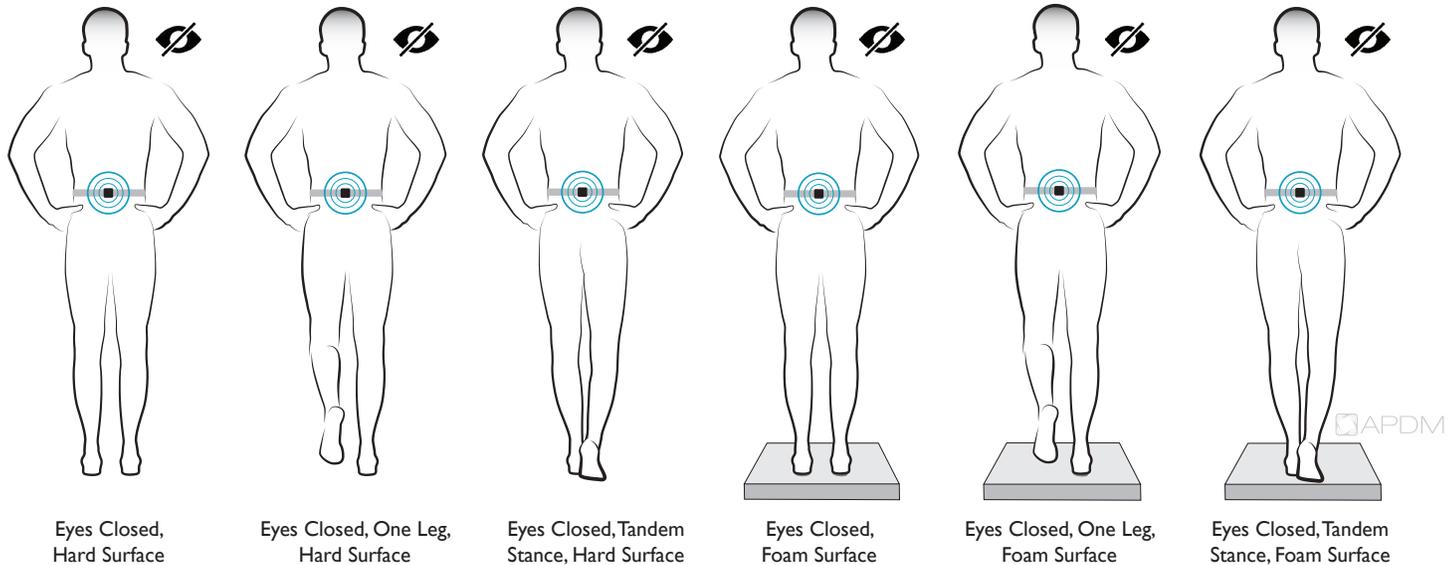

TEST MEASURES:

Postural stability in varying conditions

NUMBER OF SENSORS: 1 or 3 or 6

SETUP:

1. Have the subject fit their feet around the foot template provided with the Mobility Lab system (to normalize foot placement).

PROTOCOL:

1. Select BESS and press start test.
2. Subject should stand according to the test condition with their hands on their hips, and their eyes closed.
3. Follow the conditions outlined in the test description.
4. Press start and wait for the test to count down from 30 seconds. Move on to the next test condition.

TEST CONDITIONS:


1. Eyes Closed, Double Support, Hard Surface
2. Eyes Closed, One Leg, Hard Surface
3. Eyes Closed, Tandem Stance, Hard Surface
4. Eyes Closed, Double Support, Foam Surface
5. Eyes Closed, One Leg, Foam Surface
6. Eyes Closed, Tandem Stance, Foam Surface

NORMATIVE VALUES:

Normative values were collected following the protocol listed above.

mBESS

The modified Balance Error Scoring System (mBESS) test is a shortened version of the BESS test. It is a measure of assessing static postural stability, designed for the mild head injury population, and to assist in return to sports play decisions.

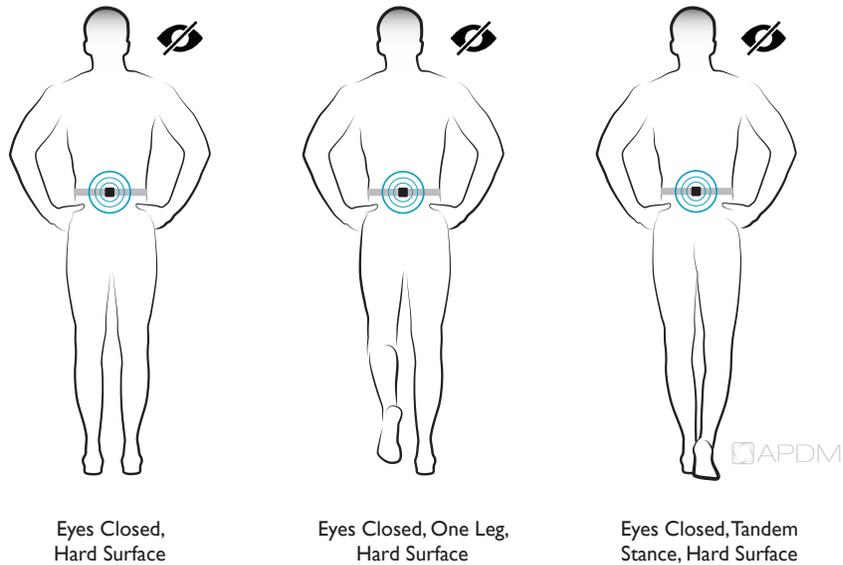

TEST MEASURES:

Postural stability in varying conditions

NUMBER OF SENSORS: 1 or 3 or 6

SETUP:

1. Have the subject fit their feet around the foot template provided with the Mobility Lab system (to normalize foot placement).

PROTOCOL:

1. Select mBESS and press start test.
2. Subject should stand according to the test condition with their hands on their hips, and their eyes closed.
3. Follow the conditions outlined in the test description.
4. Press start and wait for the test to count down from 30 seconds. Move on to the next test condition.

TEST CONDITIONS:


1. Eyes Closed, Double Support, Hard Surface
2. Eyes Closed, One Leg, Hard Surface
3. Eyes Closed, Tandem Stance, Hard Surface

NORMATIVE VALUES:

Normative values were collected following the protocol listed above.

360° TURN

The 360 degree Turn Test is a measure of dynamic balance. The subject turns in a complete circle (360 degrees) while time to complete and/or number of steps to complete the turn are recorded.

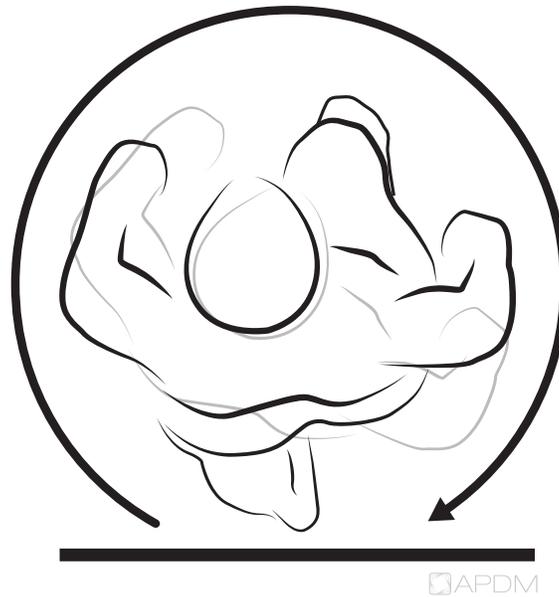

TEST MEASURES:

Turn velocity, time, number of steps

NUMBER OF SENSORS: 6

SETUP:

1. Place a piece of masking tape on the floor to mark the start/stop position. Have the subject fit their feet around the foot template provided with the Mobility Lab system (to normalize foot placement).

PROTOCOL:

1. Select 360° Turn and press start test.
2. Subject should stand with their toes aligned with the tape.
3. Press start and wait for the subject to complete a full turn. Press stop when the subject's shoulders are back in the start position.

NORMATIVE VALUES:

Normative values were collected following the protocol listed above.

5x SIT TO STAND

The 5 Times Sit to Stand (5xSTS) test is a measure of functional lower limb muscle strength. It is useful in quantifying functional change of transitional movements.



TEST MEASURES:

Trunk excursion, stand time, cadence, total time

NUMBER OF SENSORS: 6

SETUP:

1. It is preferable to use a chair with no armrests, to ensure that subjects stand without assistance.

PROTOCOL:

1. Select 5x Sit to Stand and press start test.
2. Subject should sit with their back against the back of the chair.
3. Press start and wait for the subject to stand up completely, then return to the sitting position. Press stop when the subject has returned to the sitting position the 5th time.

NORMATIVE VALUES:

Normative values were collected following the protocol listed above.

CONTACT US

info@apdm.com

503.446.4055

apdm.com/mobility

